KINAMED®
INCORPORATED

www.kinamed.com/symbols_glossary

Kinamed, Inc.
820 Flynn Road
Camarillo, CA 93012 USA
(800) 827-5775 (805) 384-2748
www.kinamed.com

©2020 Kinamed®, Inc.

B00107F
Y998-371

JP Japanese Quick Tap® Power Driver 使用説明書

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English Quick Tap® Power Driver Instructions for Use

B00107F
Y998-371

Both surgeon and staff must read this entire *Instructions for Use* document prior to the start of the surgical case as it contains essential information for successful use of the unit.

Indications for use:

The Quick Tap Power Driver is indicated for use in the installation of NeuroPro® System bone screws. Both standard self-tapping and Quick Tap self-drilling screws in the NeuroPro System may be installed with the Power Driver. Quick Tap Self-Drilling Screws generally do not require pre-drilling a pilot hole unless unusually dense bone is encountered.

CAUTION

Federal Law (USA) restricts this device to sale by or on the order of a licensed physician.

Devices labeled for single use are intended to be used once only, for a single patient, because they may not perform as intended if they are reused. Reuse may lead to failure of the device to perform as intended.

Do not use if package is damaged.

The Quick Tap Power Driver is supplied sterile-packaged and is indicated for **single-use only**.

Prior to Surgery:

Ensure that a Power Driver unit is available for the surgery. It is advisable to have at least one back-up unit available in the event that sterility is compromised on the unit in use.

The Quick Tap Power Driver is not provided with a screwdriver blade in the package. A screwdriver blade from the NeuroPro implant/instrument set must be used with the Power Driver.

During Surgery:

Carefully open the pouch containing the sterile driver and deliver the unit to the sterile field. Insert a sterilized NeuroPro screwdriver blade from the NeuroPro implant/instrument set into the Power Driver. Pull back the collet at the end of the unit using the thumb or index finger and insert the screwdriver blade into the Driver until only about one inch of blade projects out of the unit. Release the collet. If the collet does not snap back to its original position rotate the driver blade until it does snap back.

The Power Driver is designed to be held in a “pencil grip” fashion with the index finger on top of the Forward and Reverse buttons. With a proper grip both the Forward and Reverse buttons can be operated by depressing with the tip of the index finger without the need to re-adjust the grip.

CAUTION

Do not depress both Forward and Reverse buttons simultaneously. Doing so may result in overheating.

The surgical assistant or the surgeon may load the screw from the implant/instrument tray onto the driver blade while it is installed in the Power Driver in a fashion similar to that used with the standard manual screwdriver. Alternatively, the surgical assistant may use two screwdriver blades from the NeuroPro set with one Power Driver unit by handing off a blade loaded with a screw to the surgeon for placement into the Power Driver unit. The second driver blade is pre-loaded and handed off to the surgeon upon installation of the first screw.

In very dense cortical bone, self-drilling screws driven with a manual screwdriver may have a tendency to bind and break due to over-torquing. The Quick Tap Power Driver is designed to deliver torque that is just less than the breaking strength of the Quick Tap Self-Drilling Screws, thus reducing the chance of their fracture in very dense bone. Therefore, if the screws bind while driving in dense bone, then the Power Driver will stall (i.e. come to a stop). If this occurs, the Power Driver should be operated in Reverse briefly, backing the screw out slightly and freeing up packed bone chips, and then operated in Forward again to complete seating of the screw.

CAUTION

Do not immerse the Power Driver in saline or other fluids.

After Surgery:

Prior to disposal of the Power Driver, the screwdriver blade is removed from the unit and returned to the NeuroPro implant/instrument tray for reprocessing with the kit. The screwdriver blade is not a disposable item.

Disposal:

The Power Driver is intended for single-use only and is not autoclavable. The unit contains Li/MnO₂ batteries which, in household use in the US, may typically be disposed of in regular trash. Disposal regulations may vary by jurisdiction and institution type so check with the appropriate local authority to determine disposal requirements.

Sterility:

The Quick Tap Power Driver is supplied sterile packaged. The packaging including outer box and inner sterile pouch should be inspected prior to use. If damage is detected, the product should not be used.